0476

**RI.MOS. s.r.l.** viale Gramsci, 29 – 41037 MIRANDOLA (MO) ITALY – tel. +39 0535 25755 –  
fax +39 0535 25509 - e-mail: [info@rimos.com](mailto:info@rimos.com) – [www.rimos.com](http://www.rimos.com)

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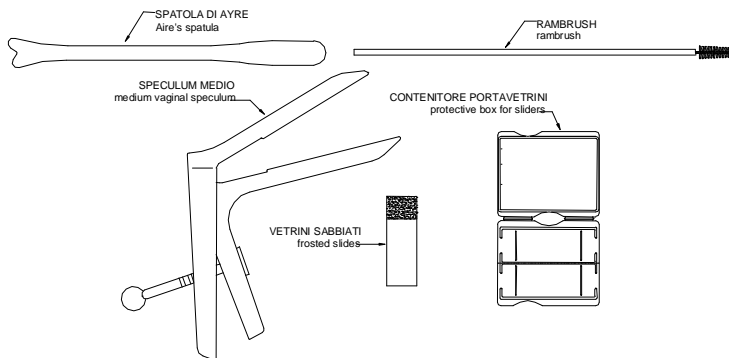
GB

#### Instructions for use

Introduce the speculum (valves closed) into the vaginal cavity. Draw the handles up close together so as to open the valves inside and dilate the cavity. Turn the central locking device horizontally to block the handles even without hand-holding. Start sampling the ectocervix, transformation zone and lower endocervix through wooden Ayres spatula. Transfer the sampled cells to a first slide and fix immediately with FISSY (fixative spray for cytology). Complete pap-test by sampling the endocervix with RAMBRUSH. Introduce the brush in the endocervical canal leaving the bristles visible. Rotate one full turn and then remove from the endocervix. Transfer the content of the brush to the second slide using a rolling or rotary motion along the surface of the slide. Fix the specimen immediately with FISSY, fixative spray for cytology. Store both the slides in the special plastic holder.

**Caution:** Be gentle with pregnant patients. Do not use Rambrush for endometrial sampling.

Read the instructions before using the device. Do not reuse. Use only if the package is not damaged. Use immediately after opening. Dispose of after use according to the laws in force. For use only under trained medical staff.

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