

Sharpsafe Label



Once you have assembled the Sharpsafe container don't forget to fill in the details on the label.

Sharpsafe Temporary Closure

To release the temporary closure feature press down on the tab, rotate the lid until it engages in the open position the container is now ready for use.



To engage the temporary closure feature release lid from the open position rotating the lid until it covers the aperture, continue to close the lid until a click is heard. This indicates the temporary closure is engaged.

Location, Location, Location



Ensure the Sharpsafe container is placed into a stand or onto a suitable surface ready for use. The container can also be placed onto a wall mounted bracket or Sharpsafe trolley depending on your requirements.